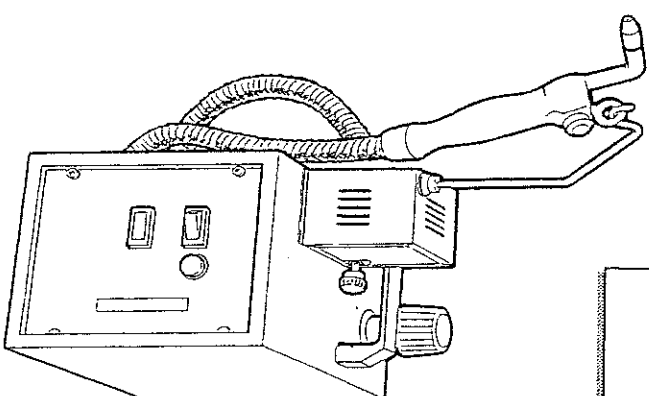
- GENERATORE DI VAPORE
- STEAM GENERATOR
- GENERATEUR DE VAPEUR
- DAMPFGENERATOR

VP Mini

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WARNINGS

- The steam generator has been designed and built for use by dentists, dental technicians and denturists. No other applications are envisaged by the Manufacturer who consequently disclaims any and all responsibility for damage of whatever kind caused by improper use of the machine.
- Modifications, tampering (however limited), and the use of non-original replacement parts automatically release the Manufacturer from any liability and shall cause the warranty to lapse.
- The machine may only be used by suitably trained adults.

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SYMBOLS

- Symbols highlighting situations requiring caution, practical tips and simple information have been used to help you read and quickly assimilate the contents of this manual.
- These symbols may be placed next to a text (and therefore refer solely to that text), next to a figure (referring to the subject shown in the figure and to the relevant text) or at the top of a page (in which case they refer to all the subjects treated on that page).

CAUTION! Pay attention to the meaning of the symbols: they are intended to avoid repeating technical concepts or safety warnings, and should therefore be regarded as reminders. Consult the chart below if you are in doubt as to their meaning.

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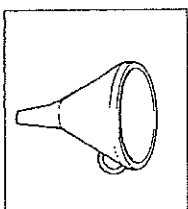
Fig. 2

PACKING

- The machine is shipped inside a cardboard box and is protected by plastic sheeting.
- The overall dimensions and weight are purely approximate.
- When the machine is delivered, check the integrity of the box.
- Remove the machine from the packing and dispose of the packing material in accordance with prevailing environmental protection regulations.

SUPPLY

- The steam generator is delivered ready for use.

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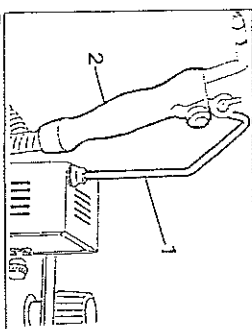
Fig. 7

INSTRUCTIONS FOR USE

- Place the rod (1) supporting the steam gun (2) in the socket.

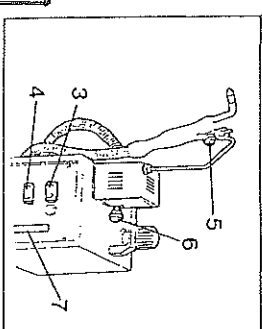
Fig. 8

- Set the switch (3) to ON.
- Wait for the "Machine ready" light (4) to go on.
- Press the push-button (5) to activate the steam supply.
- After performing a steam test, use the knob (6) to adjust the quantity required.

▲ **Point the gun and therefore the steam jet only in the direction of the job you are performing. Do not point it in any other direction!**

• While the generator is in use, check the water level on the indicator (7).

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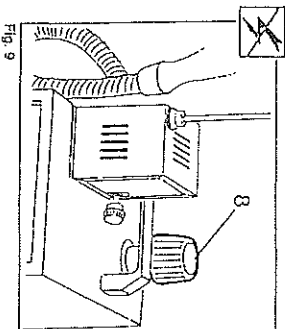
Fig. 9

FILLING WHILE THE MACHINE IS HOT

- Switch off and wait for several minutes.
- Loosen the cap (9) a quarter turn.
- Let the remaining pressure escape.
- Remove the cap carefully.
- Fill up with hot water if possible, or wait for two minutes and fill with cold water.

▲ **Turn off the machine when you have finished using it.**

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TROUBLESHOOTING

PROBLEM	POSSIBLE CAUSE	REMEDY
No steam escapes from nozzle	a) The 24 Volt transformer has become detached and the cables have worked loose (during transport).	a) Remove the plug and the control panel, check the transformer and the cables (if the weld joints have become detached, reinforce and reweld them).
	b) The push-button (5) is defective and the contact isolated.	b) Remove the plug, undo the three screws on the bakelite handgrip of the gun (2) and clean the contacts on the push-button (5). If the problem persists, replace the push-button (5).
	c) The solenoid valve is jammed.	c) Replace solenoid valve.
	d) The sprayer is clogged with calcium.	d) Proceed as described in a); loosen the pipe clamp, remove the nozzle of the gun and immerse it in diluted vinegar or another common household substance. Do not use acid.
	e) The knob (6) is closed.	e) Turn knob to open.
Steam escapes from the cap	a) The cap is not screwed on correctly.	a) Screw and tighten correctly.
	b) The inner seal is defective.	b) Replace the seal.
	c) The pressure switch is defective.	c) Replace the pressure switch.
The unit does not heat up	a) The heating element is defective.	a) Contact the Customer Service.
The unit heat very slowly	a) The boiler tank is full of calcium deposit.	a) Decalcify as described in the maintenance instructions.
	a) Is the appliance plugged in?	a) Connect up the appliance.
The water level indicator light does not come on even when the main switch is on	b) Is power supply regular?	b) Check phases.
	c) Is the fuse faulty?	c) Check the fuse and if necessary replace. If it blows again, contact Customer Service.
	d) Is the bulb faulty?	d) Replace. Pull out the plug; remove the 4 screws of the front plate, lift up and replace the bulb.

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Do not exceed maximum water level.

- Condensation inside the pipe generates a small quantity of water which will exit at the first steam request; this condition confirms correct appliance operation.
- With the boiler full of water, the volume of steam drops and a consequent fairly quick pressure drop is possible; the greater the volume of steam, the smaller the quantity of water in the boiler.

VOLUME OF WATER

- Every time the appliance is started, fill the boiler with water.
- Start the appliance and leave it under power the whole day.
- The water level must never fall below the minimum.
- In the event of lack of water, the appliance will produce a continuous acoustic alarm signal and heating will automatically stop.

DECALCIFICATION

- Depending on how hard the water is (percentage of calcium), it will be necessary to decalcify the appliance at regular intervals:

- use diluted vinegar or anti-calcium product;
- allow the vinegar or product to work all night long;
- empty the appliance and leave it upside down for a while;
- rinse the boiler;
- fill the boiler again with water.

- Some calcium deposits could remain attached to the steam distributor; blow these away using compressed air to free the syringe duct.

PRECAUTIONS

- During operation, the magnetic valve, the steam generator cover and the distributor can reach temperatures above 80° C.

SAFETY DEVICES

- The safety devices fitted on this machine are:
- cap with safety valve,
- pressure switch,
- thermostat on heating element.

In this case the acoustic alarm sounds without interruption.

- The appliance is fitted with one fuse (24 V) for safeguarding use (syringe) and with another fuse (220 V) protecting all other functions. In case of faulty operation, check these fuses and if necessary, replace with identical ones.

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